

Q-lite Part # 007961

Quantity - 1

Manufactured by: Breas Medical, Inc. N. Billerica, MA 01862

Q-lite is a trademark of Breas Medical, Inc. Patent Pending.



The Q-lite muffler enclosed in this package replaces the Qtube mentioned in the Z2 CPAP and Z2 Auto CPAP User Guides. Please see below for information specific to the Q-lite.

The Q-lite muffler was designed to help quiet the "wind" noise generated up the CPAP therapy tube. It should be installed between the end of the therapy tube and the outlet tube adapter of the Z1° or Z2° using the short (8") extension tube.

The Q-lite was designed to be lighter and less bulky than the original Qtube. Unlike the Qtube, there are no owner servicable parts to replace.

When cleaning the CPAP tube, disconnect the Q-lite. Do not wash.

Examine the Q-lite for cracks or breaks in the plastic and replace as neccessary.

In laboratory testing, the interior foam showed no sign of bacterial growth after 30 days.

